

Paw Brothers®

Professional Hybrid Cages

PBP89430C & PBP89435C



Assembly Instructions

Initial Inspection

When your Paw Brothers® Professional Hybrid Cage arrives, check immediately for signs of shipping damage. If you notice damage, save the shipping carton for claim purposes, and immediately report the problem to your supplier and to the shipping company.

Caution: Use safety gloves when handling all parts. Some edges may be sharp.

Read all instructions.

Tools required for assembly: # 2 Phillips Screwdriver and 2 hex head wrenches (included)


Parts List:

1	Door Assembly	PBP89485	PBP89486
2	Top/Bottom Panels	PBP89485-1	PBP89486-1
2	Side Panels	PBP89485-2	PBP89486-2
1	Back Panel	PBP89485-3	PBP89486-3
2	Hex Head Wrenches	PBP89478-1	
1	Bag of Self-tapping Sheet Metal Screws	PBP89478-2	
8	Hex Head Fasteners	PBP89478-3	
1	Floor Grate	PBP89465	PBP89466
1	ABS Plastic Pan	PBP89450	PBP89454

Assembly Instructions:

Step 1. Unpack all cage parts and compare all the pieces to the parts list.



Step 2. Using a #2 Phillips screwdriver, put two side panels and top/bottom panel together with sheet metal screws. The flanges face out as shown in Figure A below.

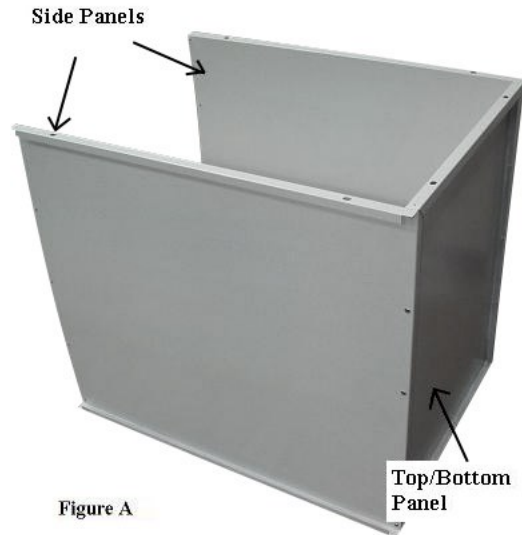

Step 3. Assemble the remaining top/bottom panel to the other panels with sheet metal screws as shown in Figure B.

Step 4. Put the assembled panels face down with the front of the cage on the ground. Assemble the back panel to the other four panels with sheet metal screws. (See Figure C)

NOTE! For proper panel alignment, do not completely tighten the screws until each hole has a screw.

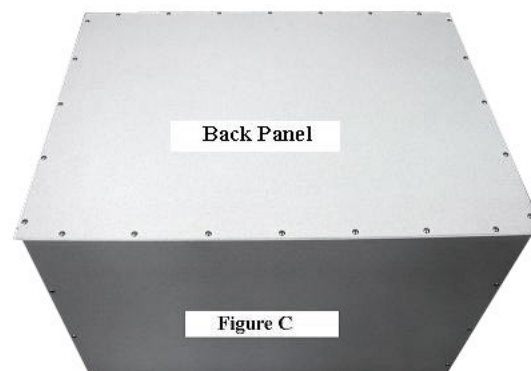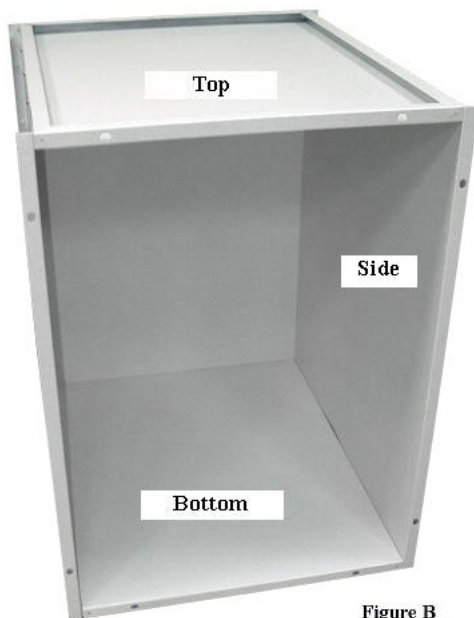

Step 6. Assemble the Door Assembly to the cage using the Hex Head Fasteners as shown in Figure E.

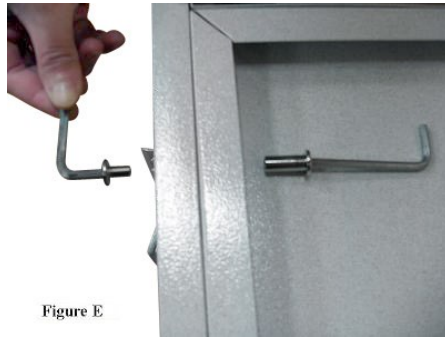


Figure E

Hex Head Fastener installation



Figure G

Step 7. Place the Floor Grate in the cage. Slide the ABS Plastic Pan underneath the grate. See Figure G.